



# SCALE UP COVID-19 ANTIBODY TESTING WITH A RAPID LATERAL FLOW FINGERSTICK IgG/IgM ASSAY

## MAKE TESTING MORE WIDELY AVAILABLE

- Antibody tests are important to confirm that an infection has occurred
- Rapid scale-up capability
- Time to result: 10–20 minutes
- Fingerstick whole blood, venous whole blood or serum/plasma sample types
- Decentralized testing can help to alleviate overcrowding in hospitals
- Useful in drive-through, home visit, point-of-care, or lab settings




Panbio™ COVID-19  
IgG/IgM Rapid Test Device


# FINGERSTICK TEST PROCEDURE

Consult Instructions for Use for complete procedure.


## 1 PUNCTURE FINGER WITH STERILE LANCET

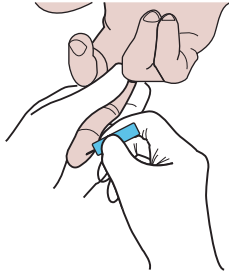

## 2 ADD 20 µL SAMPLE



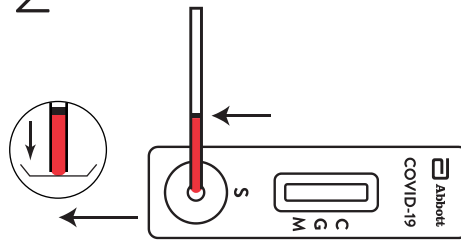


## 3 ADD 2 DROPS OF BUFFER

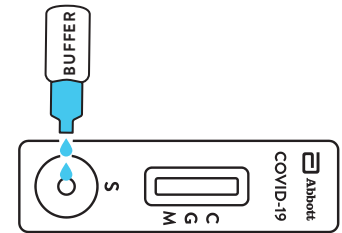

## 4 READ RESULTS AT 10 MINUTES

DO NOT READ AFTER 20 MINUTES

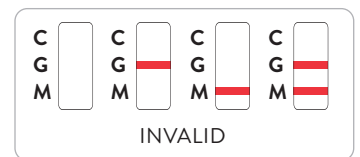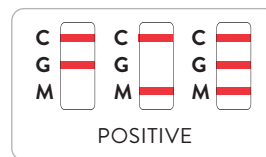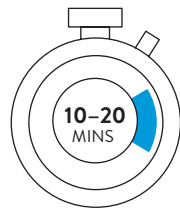

## SPECIFICATIONS

**INTENDED USE:** An *in vitro* diagnostic rapid test for the qualitative detection of IgG and IgM antibodies to SARS-CoV-2 in human serum, plasma, venous and fingerstick whole blood. For professional use only. Intended to be used as an aid in the diagnosis of SARS-CoV-2 infection. The product may be used in any laboratory and non-laboratory environment that meets the requirements specified in the Instructions for Use and local regulation. The test provides preliminary test results. Negative results will not preclude SARS-CoV-2 infection, and they cannot be used as the sole basis for treatment or other management decisions. The test is not intended to be used as a donor screening test for SARS-CoV-2.

- **TEST TIME:** 10–20 minutes
- **SENSITIVITY:** 95.8%
- **SPECIFICITY:** 94.0%
- **STORAGE:** 2°C–30°C
- **CE MARK**
- **SAMPLE TYPE:** Fingerstick whole blood, venous whole blood, serum/plasma
- **SAMPLE SIZE:**
  - 20 µL fingerstick or venipuncture whole blood
  - 10 µL serum or plasma

## ORDER INFORMATION

- **CATALOG NUMBER:** ICO-T402
- **QUANTITY:** 25 tests/kit
- **CONTENTS:** 25 Test Devices, 25 Specimen Droppers, 1 Buffer (3 mL), 1 Instructions for Use

CONTACT YOUR LOCAL REPRESENTATIVE TODAY TO DEVELOP A RAPID NATIONAL SCALE-UP PROGRAM.

Product not available in all countries.  
[Abbott.com/POCT](http://Abbott.com/POCT)



SCAN FOR MORE INFORMATION

